

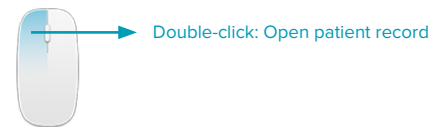
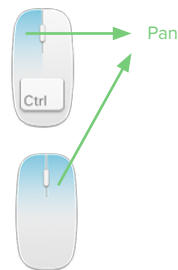


Shortcuts in DTX Studio™ Clinic 2.3 for Windows

Patient Module

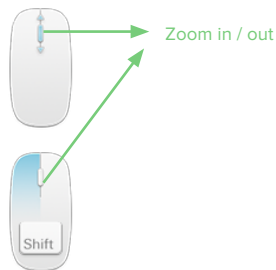

- Patient Module
- Image Preview

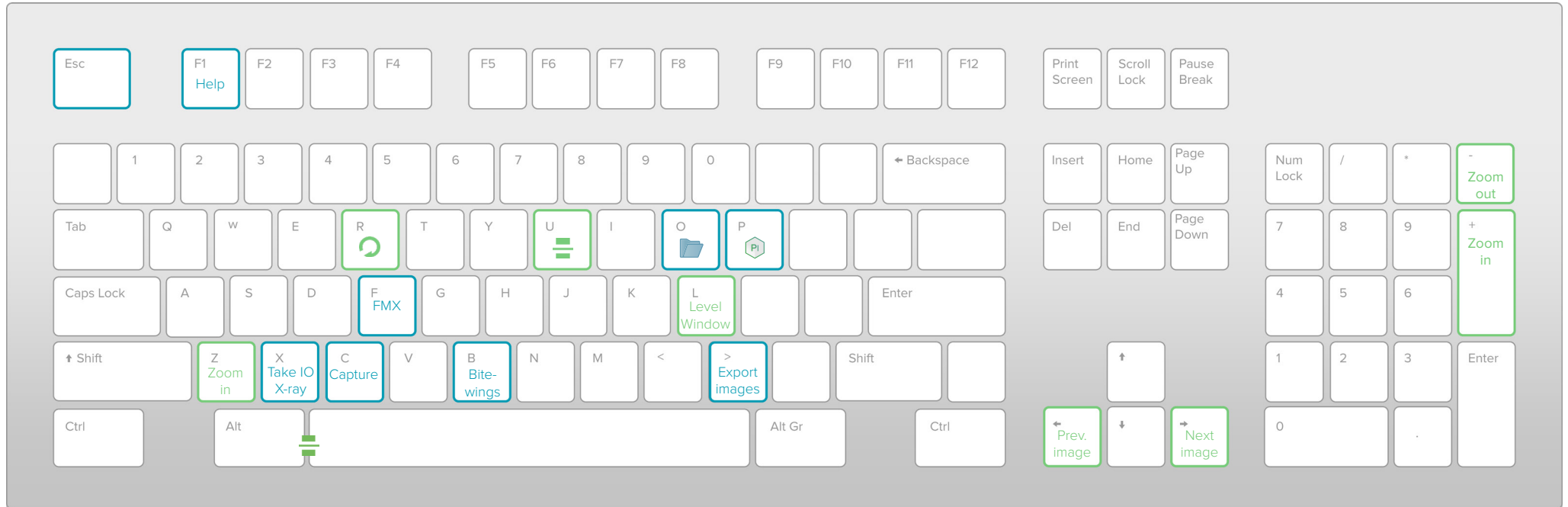

Shortcuts in DTX Studio™ Clinic 2.3 for Windows




Open Patient Record

Switch workspaces:

- | | |
|------------------------------|-----------------------|
| F2: 3D patient / Dental scan | F7: Ceph |
| F3: 3D panoramic | F8: Clinical pictures |
| F4: Tooth | F9: Implant |
| F5: Panoramic | F10: Intraoral camera |
| F6: Intraoral | F12: Recent images |





■ 2D

■ 3D

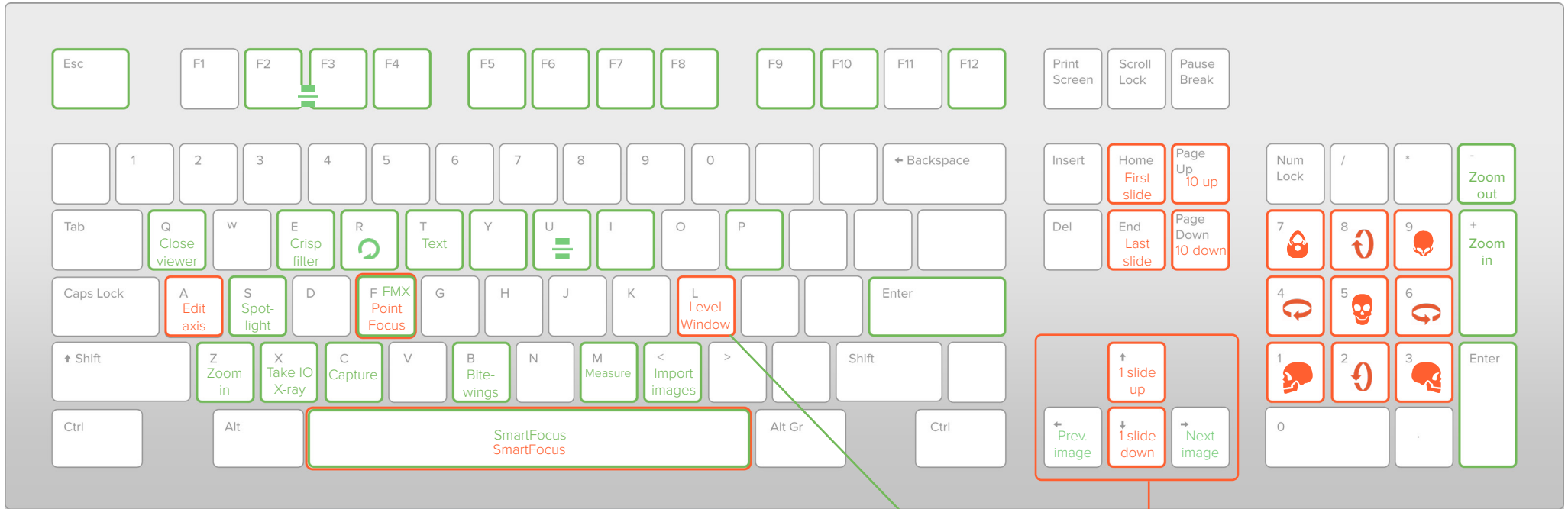

Adjust brightness and contrast

Rotate 3D view

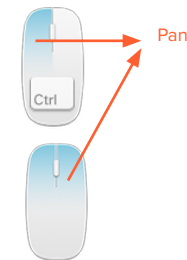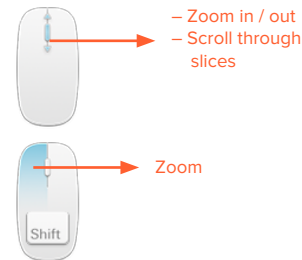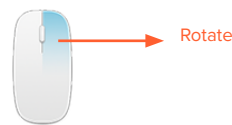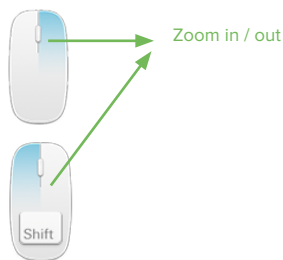


Shortcuts in DTX Studio™ Clinic 2.3 for Windows

Extended Shortcuts


Patient Module

Shortcut	Description
Esc	Close active patient
Shift + P	Export to new patient in DTX Studio™ Implant

Image Preview in Patient Module

Shortcut	Description
Esc	Close preview of the image
Ctrl + R	Reset zoom
Shift + R	Rotate 2D image counterclockwise
Shift + U	Mirror 2D image vertically 
Shift + Z	Zoom out

Open Patient Record

Shortcut	Description
Esc	– Restore down maximized window – Finish Spotlight action – Close dialog or wizard
Enter	Capture with intraoral camera (when the livestream is active)
Alt + F4	Alt + F4: Exit application
Ctrl + I	Show / Hide viewer information
Ctrl + M	Maximize / Restore down
Ctrl + P	Print viewer images and content
Ctrl + R	Reset zoom
Shift + R	Rotate image counterclockwise
Ctrl + S	Save diagnosis
Ctrl + T	Adjust tooth range
Shift + U	Mirror image vertically 
Shift + Z	Zoom out
↑ ↓ → ←	Translate the selected implant in the plane of the reslice
Ctrl + ↑ ↓ → ←	Translate the selected implant along its axis