

DTX Studio™ Clinic 2.4 Shortcuts - MacOS®



Patient Module

Esc	F1 Help	F2	F3	F4	F5	F6	F7	F8	F9	F10	F11	F12	Print Screen	Scroll Lock	Pause Break	■ Patient Module ■ Image Preview					
	1	2	3	4	5	6	7	8	9	0	← Backspace		Insert	Home	Page Up	Num Lock	/	*	-	Zoom out	
Tab	Q	W	E	R	T	Y	U	I	O	P			Del	End	Page Down	7	8	9	+	Zoom in	
Caps Lock	A	S	D	F FMX	G	H	J	K	L Level Window	Enter						4	5	6			
↑ Shift	Z Zoom in	X Take IO X-ray	C Capture	V	B Bite-wings	N	M	<	>	Export images	Shift						1	2	3	Enter	
fn	Ctrl	Option	Cmd								Cmd	Option					0			.	
													← Prev. image	↑	↓	→ Next image					



Open Patient Record

Esc	F1	F2	F3	F4	F5	F6	F7	F8	F9	F10	F11	F12	Print Screen	Scroll Lock	Pause Break	■ 2D ■ 3D					
	1	2	3	4	5	6	7	8	9	0	← Backspace		Insert	Home First slide	Page Up 10 up	Num Lock	/	*	-	Zoom out	
Tab	Q Close viewer	W	E Crisp filter	R	T Text	Y	U	I	O	P			Del	End Last slide	Page Down 10 down	7	8	9	+	Zoom in	
Caps Lock	A Edit axis	S Spot-light	D	F FMX Point Focus	G	H	J	K	L Level Window	Enter						4	5	6			
↑ Shift	Z Zoom in	X Take IO X-ray	C Capture	V	B Bite-wings	N	M Measure	<	>	Import images	Shift						1	2	3	Enter	
fn	Ctrl	Option	Cmd	SmartFocus SmartFocus							Cmd	Option					0			.	
													← Prev. image	↑ 1 slide up	↓ 1 slide down	→ Next image					

Adjust brightness and contrast

Rotate 3D view

Extended Shortcuts

Open Patient Record

Shortcut	Description
Esc	– Restore down maximized window – Finish Spotlight action – Close dialog or wizard
(fn+) F2	3D patient / Dental scan
(fn+) F3	3D panoramic
(fn+) F4	Tooth <=> Endo
(fn+) F5	Panoramic
(fn+) F6	Intraoral
(fn+) F7	Ceph
(fn+) F8	Clinical pictures
(fn+) F9	Implant
(fn+) F10	Intraoral camera
(fn+) F12	Recent images
Enter	Capture with intraoral camera (when the livestream is active)
Cmd + Q	Exit application
Cmd + I	Show / Hide viewer information
Cmd + M	Maximize / Restore down
Cmd + P	Print viewer images and content
Cmd + R	Reset zoom
Shift + R	Rotate image counterclockwise
Cmd + S	Save diagnosis
Cmd + T	Adjust tooth range
Shift + U	Mirror image vertically
Shift + Z	Zoom out
↑ ↓ → ←	Translate the selected implant in the plane of the reslice
Cmd + ↑ ↓ → ←	Translate the selected implant along its axis

Patient Module

Shortcut	Description
Esc	Close active patient
Shift + P	Export to new patient in DTX Studio Implant

Image Preview in Patient Module

Shortcut	Description
Esc	Close preview of the image
Cmd + R	Reset zoom
Shift + R	Rotate 2D image counterclockwise
Shift + U	Mirror 2D image vertically
Shift + Z	Zoom out

Mouse Interactions

Patient List



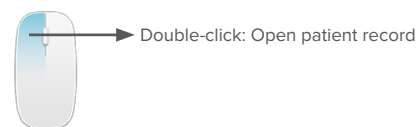


Image Preview in Patient Module

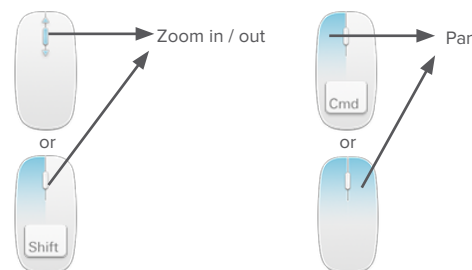

2D - Open Patient Record

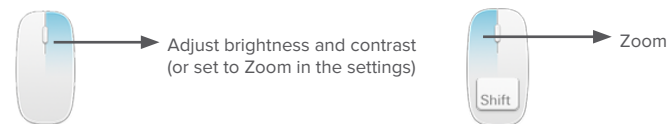

3D - Open Patient Record

