

# DTX Studio™ Clinic 2.4 Shortcuts - Windows®



## Patient Module

Esc	F1 Help	F2	F3	F4	F5	F6	F7	F8	F9	F10	F11	F12	Print Screen	Scroll Lock	Pause Break	<span style="color: blue;">■</span> Patient Module <span style="color: green;">■</span> Image Preview				
	1	2	3	4	5	6	7	8	9	0	← Backspace		Insert	Home	Page Up	Num Lock	/ *	- Zoom out		
Tab	Q	W	E	R	T	Y	U	I	O	P			Del	End	Page Down	7	8	9	+ Zoom in	
Caps Lock	A	S	D	F FMX	G	H	J	K	L Level Window	Enter						4	5	6		
↑ Shift	Z Zoom in	X Take IO X-ray	C Capture	V	B Bite-wings	N	M	<	> Export images	Shift				↑	1	2	3	Enter		
Ctrl	Alt							Alt Gr	Ctrl			← Prev. image	↓	→ Next image			0	.		



## Open Patient Record

Esc	F1	F2	F3	F4	F5	F6	F7	F8	F9	F10	F11	F12	Print Screen	Scroll Lock	Pause Break	<span style="color: green;">■</span> 2D <span style="color: red;">■</span> 3D					
	1	2	3	4	5	6	7	8	9	0	← Backspace		Insert	Home First slide	Page Up 10 up	Num Lock	/ *	- Zoom out			
Tab	Q Close viewer	W	E Crisp filter	R	T Text	Y	U	I	O	P			Del	End Last slide	Page Down 10 down	7	8	9	+ Zoom in		
Caps Lock	A Edit axis	S Spot-light	D	F FMX Point Focus	G	H	J	K	L Level Window	Enter						4	5	6			
↑ Shift	Z Zoom in	X Take IO X-ray	C Capture	V	B Bite-wings	N	M Measure	< Import images	>	Shift						1	2	3	Enter		
Ctrl	Alt	SmartFocus SmartFocus						Alt Gr	Ctrl			← Prev. image	↑ 1 slide up	↓ 1 slide down	→ Next image			0	.		

Adjust brightness and contrast

Rotate 3D view

## Extended Shortcuts

### Open Patient Record

Shortcut	Description
Esc	– Restore down maximized window – Finish Spotlight action – Close dialog or wizard
F2	3D patient / Dental scan
F3	3D panoramic
F4	Tooth <=> Endo
F5	Panoramic
F6	Intraoral
F7	Ceph
F8	Clinical pictures
F9	Implant
F10	Intraoral camera
F12	Recent images
Enter	Capture with intraoral camera (when the livestream is active)
Alt + F4	Exit application
Ctrl + I	Show / Hide viewer information
Ctrl + M	Maximize / Restore down
Ctrl + P	Print viewer images and content
Ctrl + R	Reset zoom
Shift + R	Rotate image counterclockwise
Ctrl + S	Save diagnosis
Ctrl + T	Adjust tooth range
Shift + U	Mirror image vertically
Shift + Z	Zoom out
↑ ↓ → ←	Translate the selected implant in the plane of the reslice
Ctrl + ↑ ↓ → ←	Translate the selected implant along its axis

### Patient Module


Shortcut	Description
Esc	Close active patient
Shift + P	Export to new patient in DTX Studio Implant

### Image Preview in Patient Module


Shortcut	Description
Esc	Close preview of the image
Ctrl + R	Reset zoom
Shift + R	Rotate 2D image counterclockwise
Shift + U	Mirror 2D image vertically
Shift + Z	Zoom out

## Mouse Interactions


### Patient List

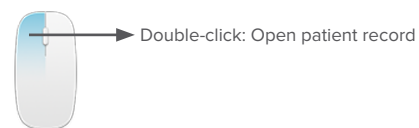

### Image Preview in Patient Module

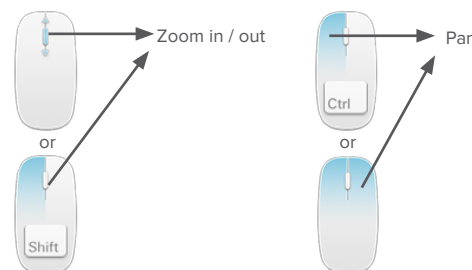

### 2D - Open Patient Record

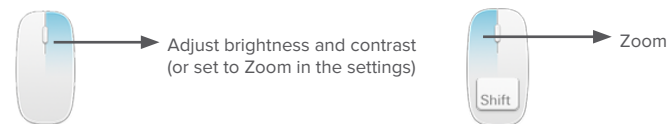

### 3D - Open Patient Record

