

DTX Studio™ Clinic 3.1 Shortcuts - MacOS®



Patient Module

| | | | | | | | | | | | | | | | | | | | | | |
|-----------|-----------|-----------------|-----------|-------|--------------|----|----|----|-----------------|-------|-------------|-----|---------------|-------------|-------------|--|-----|------------|-----------|--|--|
| Esc | F1 Help | F2 | F3 | F4 | F5 | F6 | F7 | F8 | F9 | F10 | F11 | F12 | Print Screen | Scroll Lock | Pause Break | ■ Patient Module ■ Image Preview | | | | | |
| | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 | 9 | 0 | ← Backspace | | Insert | Home | Page Up | Num Lock | / * | - Zoom out | | | |
| Tab | Q | W | E | R | T | Y | U | I | O | P | | | Del | End | Page Down | 7 | 8 | 9 | + Zoom in | | |
| Caps Lock | A | S | D | F FMX | G | H | J | K | L Level Window | Enter | | | | | | 4 | 5 | 6 | | | |
| ↑ Shift | Z Zoom in | X Take IO X-ray | C Capture | V | B Bite-wings | N | M | < | > Export images | Shift | | | | | | 1 | 2 | 3 | Enter | | |
| fn | Ctrl | Option | Cmd | | | | | | | Cmd | Option | | | | | 0 | | | . | | |
| | | | | | | | | | | | | | ← Prev. image | ↑ | ↓ | → Next image | | | | | |



Open Patient Record

| | | | | | | | | | | | | | | | | | | | | | |
|-----------|----------------|-----------------|----------------|-----------------------|--------------|----|-----------|-----------------|----------------|-------|-------------|-----|---------------|------------------|-------------------|--|-----|------------|-----------|--|--|
| Esc | F1 | F2 | F3 | F4 | F5 | F6 | F7 | F8 | F9 | F10 | F11 | F12 | Print Screen | Scroll Lock | Pause Break | ■ 2D ■ 3D | | | | | |
| | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 | 9 | 0 | ← Backspace | | Insert | Home First slide | Page Up 10 up | Num Lock | / * | - Zoom out | | | |
| Tab | Q Close viewer | W | E Crisp filter | R | T Text | Y | U | I | O | P | | | Del | End Last slide | Page Down 10 down | 7 | 8 | 9 | + Zoom in | | |
| Caps Lock | A Edit axis | S Spot-light | D | F FMX Point Focus | G | H | J | K | L Level Window | Enter | | | | | | 4 | 5 | 6 | | | |
| ↑ Shift | Z Zoom in | X Take IO X-ray | C Capture | V | B Bite-wings | N | M Measure | < Import images | > | Shift | | | | | | 1 | 2 | 3 | Enter | | |
| fn | Ctrl | Option | Cmd | SmartFocus SmartFocus | | | | | | Cmd | Option | | | | | 0 | | | . | | |
| | | | | | | | | | | | | | ← Prev. image | ↑ 1 slide up | ↓ 1 slide down | → Next image | | | | | |

Adjust brightness and contrast

Rotate 3D view

Extended Shortcuts

Open Patient Record

| Shortcut | Description |
|----------|-------------|
|----------|-------------|

General Shortcuts

| | |
|---------|--|
| Esc | – Restore down maximized window – Finish Spotlight action – Close dialog or wizard |
| Cmd + Q | Exit application |
| Cmd + Y | Redo |
| Cmd + Z | Undo |
| Cmd + S | Save diagnosis |

Capture

| | |
|-------|---|
| Enter | Capture with intraoral camera (when the livestream is active) |
|-------|---|

Viewer Options

| | |
|---------|---------------------------------|
| Cmd + I | Show / Hide viewer information |
| Cmd + M | Maximize / Restore down |
| Cmd + P | Print viewer images and content |

Object Interactions

| | |
|-------------------|--|
| Backspace, or Del | Delete |
| ↑ ↓ → ← | Translate the selected implant in the plane of the reslice |
| Cmd + ↑ ↓ → ← | Translate the selected implant along its axis |

Adjust Image

| | |
|--------------------------|-------------------------------|
| Shift + Z | Zoom out |
| Cmd + R | Reset zoom |
| Option + → | Rotate image clockwise |
| Shift + R, or Option + ← | Rotate image counterclockwise |
| Shift + U | Mirror image vertically |
| Cmd + T | Adjust tooth range |

Workspaces

| | |
|-----------|--------------------------|
| (fn+) F2 | 3D patient / Dental scan |
| (fn+) F3 | 3D panoramic |
| (fn+) F4 | Tooth <=> Endo |
| (fn+) F5 | Panoramic |
| (fn+) F6 | Intraoral |
| (fn+) F7 | Ceph |
| (fn+) F8 | Clinical pictures |
| (fn+) F9 | Implant |
| (fn+) F10 | Intraoral camera |
| (fn+) F12 | Recent images |

Intraoral Workspace

| | |
|------------------|---|
| Shift + Ctrl + F | Change layout to default intraoral workspace template |
| Shift + Ctrl + C | Change layout to default capturing template |
| Shift + Ctrl + B | Change layout to vertical bitewing template |

Patient Module

| Shortcut | Description |
|-------------|---|
| Esc | Close active patient |
| Shift + P | Export to new patient in DTX Studio Implant |
| Option + F4 | Hide DTX Studio Clinic |

Image Preview in Patient Module

| Shortcut | Description |
|-----------|----------------------------------|
| Esc | Close preview of the image |
| Cmd + R | Reset zoom |
| Shift + R | Rotate 2D image counterclockwise |
| Shift + U | Mirror 2D image vertically |
| Shift + Z | Zoom out |

Mouse Interactions

Patient List



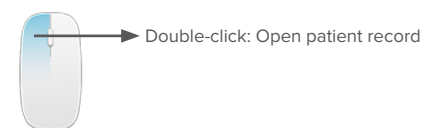


Image Preview in Patient Module

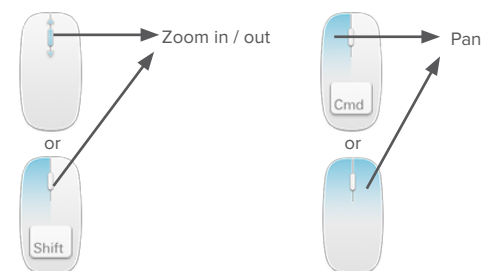

2D - Open Patient Record

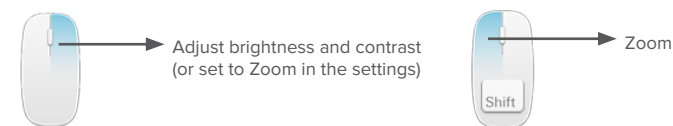

3D - Open Patient Record

