



How to Subscribe to 2D AI Dental Findings

1

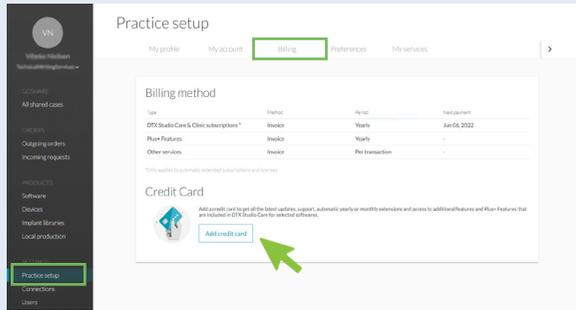


1. Log in to DTX Studio Go (go.dtxstudio.com).
2. Go to **Practice Setup > Billing**
3. Click **Add Credit card**.

2

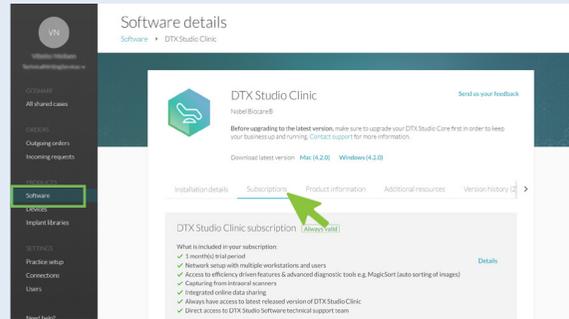


Go to **Software > Subscriptions**.

3

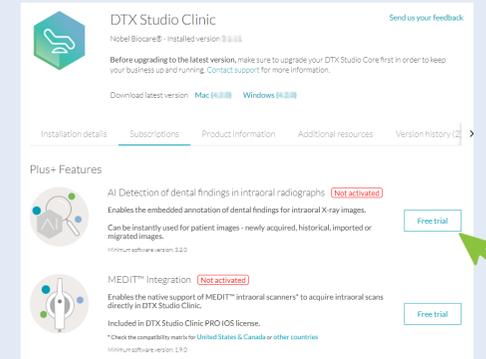


Click **Free Trial** to activate the subscription (60 days without charges).

4

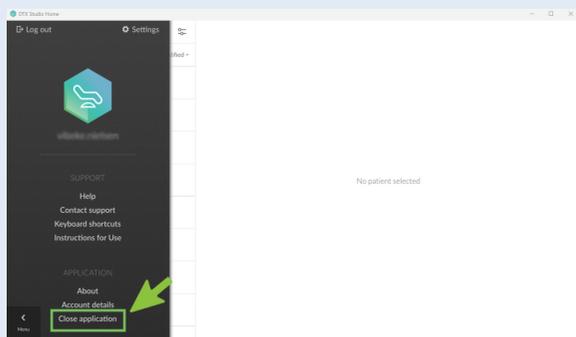


Close DTX Studio Clinic via **Menu > Close application** and start DTX Studio Clinic again for the subscription to take effect.

5

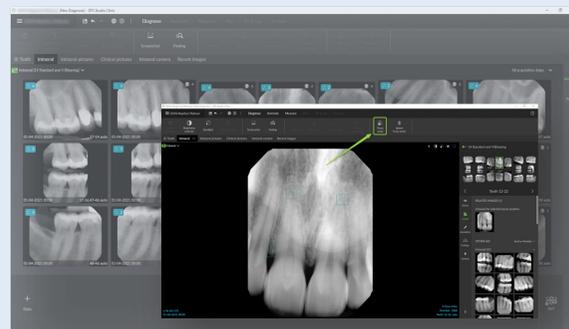

2D AI dental findings detection starts automatically for newly acquired, imported, or migrated images. It can be manually triggered individually for existing images.

6

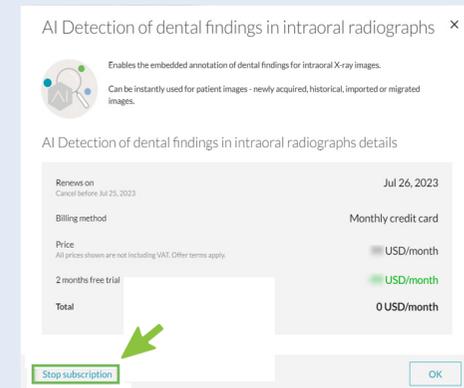

If needed, **Stop subscription** without any charges within the 60-days free trial period, or any time later.